


# CALIBRATION

**This scale is factory calibrated and does not require calibration prior to use.**

The calibration procedure should be performed using a known / certified weight in pounds. The 3102KL-AM requires 300 pounds for calibration. If you prefer to calibrate in kilograms, please call Customer Service at 1-800-815-6615.

ACTION	DISPLAY SHOWS
Press the  button to turn on the scale and while “START” is displayed, press <b>REWEIGH - ZERO - ZERO - REWEIGH</b>	“CAL”
Press <b>ENTER</b> to start the calibration procedure.	“Load/300.0”
Load 300lb onto the scale platform, then press <b>ENTER</b> .	“CLEAR”
After “CLEAR” appears on the display, remove the weight from the platform, then press <b>ENTER</b> . The scale will then calibrate zero.	“-----”
When zero calibration is complete, the display will ask for 300lb weight again.	“PUT/300.0”
Load 300lb onto the platform, then press <b>ENTER</b> .	“DONE”/“CLEAR”
After “CLEAR” appears on the display, remove the weight from the platform and the scale will reboot.	“-----”
Scale is ready for normal operation after reboot and the display returns to “0.0”.	“0.0”