Model 553KL / 553KG

Find the date code on your scale's product label and follow this chart to select the version specific user instructions for your scale.

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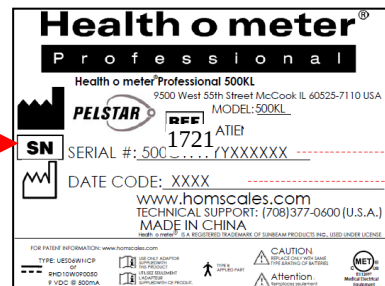
<u>Date Codes</u>	<u>Link/Page #</u>
3824, 3324, 3224, 2724, 2224 4224, 4724, 5124, all date codes ending in 25	Click here or skip to page 2
For all other date codes not shown above, click here or skip to page 5	

Date code can be found on the product label on the back of the display head.

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CALIBRATION

Your scale has been factory calibrated and does not require calibration prior to use. There are no requirements for calibration in the field; users should adhere to their institution's calibration policies. If required, the scale can be calibrated. A 5kg weight is required for calibration. If the scale is Everlocked into pounds, the calibration procedure will still require use of a 5kg weight.

Note: Avoid scale placement near heating/cooling vents, heating appliances, areas of high airflow or direct sunlight. Calibration must be done in a controlled environment.

Note: Only weights certified and traceable to national standards should be used for calibration procedures.

1. Press and hold **HOLD/RELEASE**, then press and release the  button to power on the scale. Remain pressing **HOLD/RELEASE** until "CAL" appears on the display. Release all buttons.
2. Once the buttons are released the following message will display, "CAL99", "CAL98" etc., until the display counts down to zero. This allows the load cells time to settle before calibration starts. **HOLD/RELEASE** can be pushed at any time to cancel the countdown, however this is not recommended.
3. Once the time has elapsed, "2Ero SCALE" will scroll horizontally across the screen. With no weight on the scale, press **ZERO/CLEAR**.
4. The scale will zero and "PLACE 5 ON SCALE PrESS HOLD" will scroll across the screen.
5. Place a certified 5kg weight on the center of the weighing tray, then press **HOLD/RELEASE**. "CAL" will flash on the screen while the scale is calibrating.
 - A. If the weight is within calibration range, a five digit number will be displayed for approximately 5 seconds then display the calibration weight. Remove the weight from the scale and the scale will restart.
 - B. If the weight placed on the scale does not match the correct calibration standard (5 kg), "Out OF RANGE" will appear on the display. Calibration is not performed. Press **ZERO/CLEAR** to exit calibration mode. Obtain the correct amount of calibration weight and begin the calibration process again.

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CALIBRATION

This scale has been factory calibrated, and does not require calibration prior to use. If required, the scale can be calibrated. Calibration of your scale is performed using kilograms (KG). 20 kilograms total are required for calibrating, applied in 5 kg increments.

Note: Only weights certified and traceable to national standards should be used for calibration procedures.

ACTION	DISPLAY SHOWS
With the scale off, press and hold the ZERO/CLEAR as well as the kg/LB button. While holding down both buttons, momentarily press the ON/OFF button to power on the scale. When "CAL" appears on the display, release all buttons. Numbers will appear indicating that you can calibrate.	"CAL" followed by "XXXXX"
Press the ZERO/CLEAR button. "C-5" will flash on the display and then return to "0."	"C-5" then "0"
Place 5 kg on the scale and when the calibration number stabilizes, press the ZERO/CLEAR button. "C-10" will flash on the display and then the display will return to "0".	"C-10" then "0"
Place an additional 5 kg on the scale for a total of 10 kg. When the calibration number stabilizes, press the ZERO/CLEAR button. "C-15" will flash on the display and then the display will return to "0".	"C-15" then "0"
Place an additional 5 kg on the scale for a total of 15 kg. When the calibration number stabilizes, press the ZERO/CLEAR button. "C-20" will appear on the display, followed by "0"	"C-20" then "0"
Place an additional 5 kg on the scale for a total of 20 kg. When the calibration numbers stabilize, press the ZERO/CLEAR button. "END" will appear on the display and the scale will automatically shut off.	"END"

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