BEFORE USING SCALE

1. Remove the isolating tab from the battery compartment.
2. The 880KL default mode is pounds. To select a different weight unit, use a pen or stylus to slide the switch on the bottom of the scale to select the desired weight unit.
3. Position scale on a firm flat surface.

OPERATING YOUR SCALE

1. Tap scale with foot to initialize. Scale is initialized using vibration. Standing on top of the platform will not initialize the scale.
2. Scale is ready when "0.0" is displayed.
3. Step on scale platform and stand still for approximately 5 seconds to lock in the weight measurement.
4. Weight measurement will flash on screen. Step off the scale.
5. Screen will hold and display the last captured weight for 10 seconds and then scale will power off.

WARNING INDICATORS

- Lo** Replace battery
ERR Weight exceeds maximum capacity of 500lb / 220kg

ADVICE FOR USE AND CARE

- Placing your scale on a hard, even floor will ensure the greatest accuracy and consistency.
- Clean your scale with a damp cloth. Do not use chemical cleaning agents.
- Do not allow your scale to become saturated with water as this can damage the electronics.
- Treat your scale with care – it is a precision instrument. Do not drop it or jump on it. It is not shock proof.

TROUBLESHOOTING GUIDE

If you experience any difficulty in the use of your scale:

- Check to make sure the batteries are correctly fitted.
- Make sure the scale is on a flat, level floor and not touching a wall.
- When you use the scale, if there is no display at all or if "Lo" is displayed – try new batteries.
- For technical assistance, contact Customer Service at 1-800-815-6615.

CHANGING THE BATTERIES

- When the symbol "Lo" appears, the batteries need to be replaced.
- Use ONLY the recommended CR2032 3.0 V Lithium button type batteries. 2 batteries are required for use.
- Remove battery cover on the underside of the scale by pushing down on the tab and lifting the cover off of the unit.
- Make sure that the new batteries are inserted with the positive polarity up! Ensure that the batteries are securely fitted into place. Replace battery cover.
- Remove the batteries if the scale will not be used for extended periods of time.
- Batteries contain substances that may be harmful to the environment and human health.
- **WARNING:** Discard old batteries carefully, out of reach of young children. Swallowing a battery may be fatal. If a battery is swallowed, contact the hospital immediately to have it removed.

This symbol indicates that batteries should not be disposed of in the normal household waste but should be taken to an appropriate local recycling facility when one is available.

Never throw batteries in a fire or attempt to open outer casings, they may explode!

MADE IN HUNGARY

LIMITED WARRANTY

What does the Warranty Cover?

Health o meter® Professional scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see “To Get Warranty Service”, below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer’s sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or in any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alterations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC
Attention R/A# _____
Return Department
9500 West 55th St.
McCook, IL 60525

If your scale is not covered by warranty, or has been damaged, an estimate of replacement costs will be provided to you for approval, prior to replacing.

PELSTAR, LLC
9500 West 55th St • McCook, IL 60525 • USA
1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC. We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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© Pelstar, LLC 2017